

# HOW TO IMPORT NOXTURNAL SETTINGS

## 1. Import/Export Settings

Your Temple Healthcare representative may have sent you a settings file (.ndb file) that has been configured to your preferences. Save this file in a temporary location e.g. Desktop

Open Noxturnal >>> Tools >>> Import/Export Settings >>> Import (top left) >>> Load Database File and broese/find the location of your saved settings file in your temporary location >>> Select All >>> Import (bottom right).

If successful, your preferred settings will now be available.

*NOTE: You may need to close Noxturnal and re-open to see the imported setting changes*

