

# Nox Maintenance Checklist

This checklist serves for your information and documentation of routine checks and maintenance activities to be performed regularly when using the Nox Medical PSG Monitoring System and its Accessories.


To guarantee continuous performance, reliability and safety of the Nox PSG, routine checks and maintenance procedures (including cleaning/disinfection) as well as damage checks should be performed regularly.

NOTE: Please apply any site specific infection control and maintenance procedures.

## OVERVIEW

	Nox A1	Consumables and Accessories
<b>Daily: Before and After Use</b>	<b>Clean and disinfect</b> entire surface with 70% -90% Isopropyl Alcohol Wipes	<p><b>Before and After Use: Visually Inspect</b> ALL components for damage</p> <p><b>After Use:</b>  <b>Clean and disinfect</b> ALL leads, electrodes, straps, oximeter probe, Nonin WristOx Unit with 70-90% Isopropyl Alcohol</p> <p><b>Dispose</b> RIP belts, nasal cannula and Filter, and disposable oximeter wrist band (if using)</p> <p><b>Batteries:</b> 1 x AA Nox (Rechargeable 2700mAh or Lithium) and 2 x AAA WristOx (Alkaline, Rechargeable or Lithium)</p> <p><b>Battery Charger:</b> Slow/Soft or Default charge activated</p>
<b>Weekly</b>	<b>Clean and disinfect</b> entire surface with 70% -90% Isopropyl Alcohol	<p><b>Clean and disinfect</b> ALL components with 70% -90% Isopropyl Alcohol</p> <p><b>Visually Inspect</b> ALL components for damage</p>
<b>Monthly</b>	<p><b>Clean and disinfect</b> entire surface with 70% -90% Isopropyl Alcohol</p> <p><b>Visual Inspection</b> of the monitor for damage</p> <p><b>RESET Device</b> Note: This will clear ALL memory. Ensure ALL data has been safely downloaded BEFORE resetting device. Any paired devices e.g. Nonin WristOx BDA numbers will need to be re-entered during the next device configuration.</p>	<p><b>Clean and disinfect</b> entire surface with 70% -90% Isopropyl Alcohol</p> <p><b>Visual Inspection</b> of the monitor for damage</p> <p><b>Batteries and Charger</b> Condition batteries once a month NOTE: This can take up to 24 hours. After conditioning is complete, ensure batteries remain on a Slow/Soft or Default Charge.</p>
	Check consumables monthly and <b>replace any expired products.</b>	

<p><b>Every 6 months</b></p>	<p><b>Visual Inspection</b> of the monitor for damage</p> <p>Check Nox A1 is running the latest version of firmware available</p> <p>Check Noxturnal is running the latest version available</p>	<p><b>Batteries</b> Dispose and Replace ALL rechargeable batteries</p> <p><b>Visually Inspect</b> ALL “Semi-Consumables” for damage. Place order for 1 x spare RIP adapter cable, cables/leads and recorder clips per device.</p>
<p><b>Recommended annually</b></p>	<p><b>Visual Inspection</b> of the monitor for damage</p> <p><b>Visually Inspect</b> ALL “Semi-Consumables” for damage. Place order for 1 x spare RIP adapter cable, cables/leads, recorder clips, battery Lid for Nox and WristOx, Oximeter Wrist band and Oximeter Soft Probe per device.</p> <p><b>Warranty</b> Contact your Temple Healthcare Representative for extended warranty/service contract options.</p>	
<p>Temple Healthcare Pty Ltd  PO Box 756, Mittagong, 2575, NSW  Ph: (02) 48580690 Fax: (02) 84580700 Email: info@templehealthcare.com.au ABN: 18 142 790 270</p>		

 **Do not use the device if any problems are noted. Replace defective or expired parts.**